

SURMODICS VASCULAR DEVICE SOLUTIONS

.035" LLMB

Large Lumen Molding Balloon Catheters

.035"
LLMB


**SECURE YOUR
VASCULAR
REAL ESTATE**

A WORKHORSE ONE-SIZE-FITS-ALL
SOLUTION FOR YOUR STENT
GRAFT DEPLOYMENT NEEDS



The workhorse solution...


- Durable compliant balloon designed to tackle multiple dilation needs
- Rapid inflate/deflate speeds procedure time and quickly restores flow
- Flexible catheter tracks anatomy with ease
- One device designed for stent grafts from 10mm to 45mm diameters
- Low profile – 12Fr sheath compatible

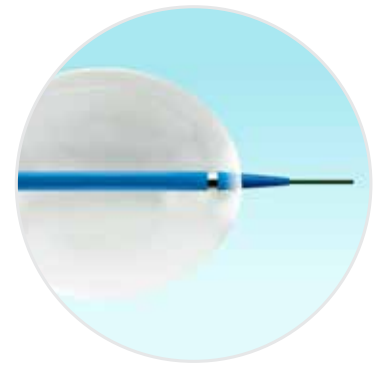

...for successful stent graft deployments

- Achieving vessel/wall apposition to secure and seal stent grafts
- Compliant balloon material designed to smooth entire graft

Product Features

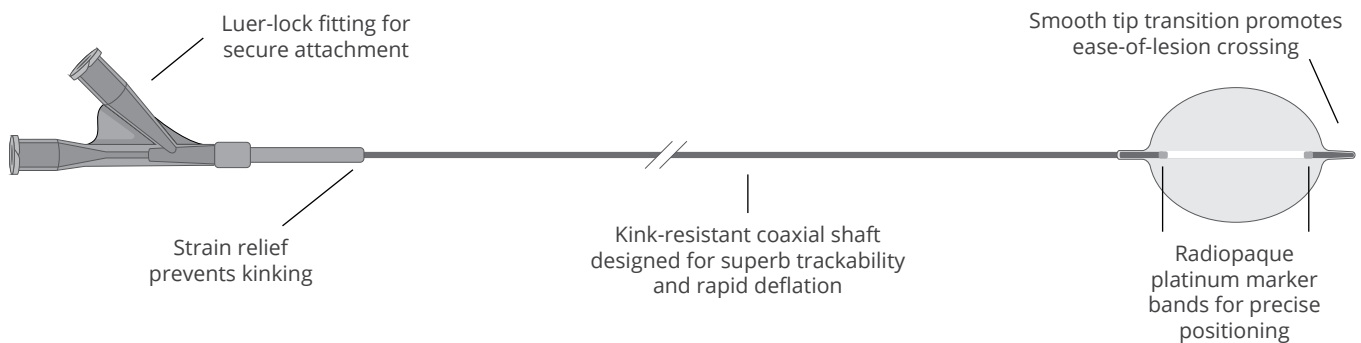

Balloon Compliance Chart

Recommended Inflation Volume	Balloon Diameter
2ml (cc)	10mm
7ml (cc)	20mm
20ml (cc)	30mm
48ml (cc)	40mm
65ml (cc)	45mm*

*Maximum Inflation Diameter -- **DO NOT EXCEED** 45mm Balloon Diameter and **DO NOT EXCEED** 65ml (cc) inflation at 45mm balloon diameter.
CAUTION: The above chart is only a guide. Balloon expansion should be carefully monitored with the use of fluoroscopy.

Product Specifications

Balloon Diameter Range	10mm - 45mm
Inflation Volume Range	2ml - 65ml
Balloon Construction	Compliant Polyurethane
Catheter Size	8Fr
Sheath Compatibility	12Fr
Catheter Length	100cm & 65cm
Recommended Guide Wire	.035"



Creagh Medical and NorMedix are now part of **Surmodics, Inc.**

9924 West 74th Street
Eden Prairie, MN 55344 USA
Toll Free 1-866-787-6639
Phone 952-500-7000
Fax 952-500-7001
www.surmodics.com

© 2016 Surmodics, Inc.
All rights reserved. 35LLMB0416 01

FOR MORE INFORMATION ON HOW **SURMODICS** CAN ENHANCE YOUR BUSINESS, PLEASE CONTACT US AT **952-500-7000** OR VISIT **WWW.SURMODICS.COM**.